autotest GLUTEN®

FRANÇAIS

ENGLISH

DEUTSCH

ITALIANO

ESPAÑOL

NEDERLANDS

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CE
CE mark

Store at between
+10°C and +27°C

Caution, running buffer: contains sodium azide ≤ 0,09%; toxic if ingested. In contact with an acid liberates a highly toxic gas. Harmful to aquatic organisms, causing long-term adverse effects. Contains gentamicin sulfate. Can cause an allergic reaction. Avoid all contact with eyes, skin or clothing. Collect spillage.

AAZ-LMB
43 rue de Bellevue
92100 Boulogne-Billancourt - France

CE 0537

AAZ.AG.14-A (20191212) User instructions for autotest GLUTEN®

FRANÇAIS

ENGLISH

DEUTSCH

ITALIANO

ESPAÑOL

NEDERLANDS

- autotest **GLUTEN**[®] is a home test for detecting coeliac disease from a sample of blood taken from the fingertip.
- autotest **GLUTEN**[®] is a single-use in vitro diagnostic device.
- autotest **GLUTEN**[®] is designed for use by individuals in a non-medical environment.
- Please read the instructions carefully and thoroughly before starting the test.
- Take the test in a well-lit place. You will need a watch or timer.

KIT CONTENTS

A Sachet

B Test cassette

C Vial of diluent

D Plastic tube containing

E a glass capillary tube

F Sterile swab

G Lancet

H Pipette

I Plaster

THE HOME TEST STEP BY STEP

Further information and a demonstration video are available on autotest-sante.com

STEP 1

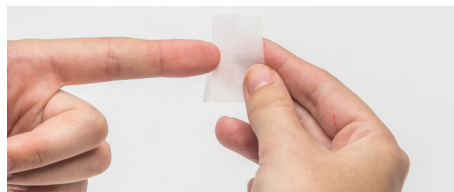
1. Open the sachet **A** and take out the test cassette **B**. Place it on a clean, flat surface.

2. Unscrew the cap from the vial of diluent **C**, open the plastic tube **D** and put down the capillary tube **E** within reach.

STEP 2

1. Wash your hands with soap and hot water then dry them before going on to the next step.
2. Remove the sterile swab **F** from its sachet, sterilize the fingertip and wait for your finger to dry.

3. Take the lancet **G** and remove its clear cap. Apply its red side to your finger and press hard until you feel the needle going in.

4. Press the fingertip to form a large drop of blood for the sample.

STEP 3

1. Take the capillary tube **E** and place it horizontally in the drop of blood until it is completely full.

2. Put the filled capillary tube **E** in the vial of diluent **C** and close it with its cap.

3. Shake the vial of diluent **C** by inverting several times until the blood from the capillary tube **E** is completely mixed in with the diluent.

4. Remove the cap from the vial of diluent **C** again and extract the diluted sample using the pipette **H** by pressing its bulb.

5. Hold the pipette **H** vertically and remove 3 drops of the diluted sample in the sample well **S** of the test cassette **B**.

STEP 4

1. Make a note of the time and wait for 5 minutes before reading the result.
2. Put on the plaster **I**.

The diagram illustrates the timing and interpretation of results. It shows a sequence of three elements: a test cassette, a clock face, and the text 'Interpretation of results (see overleaf)'. The clock face shows a 5-minute interval. Below the diagram, a red banner contains the text 'DO NOT READ BEYOND 10 MIN'.

Interpretation of results
(see overleaf)

DO NOT READ BEYOND 10 MIN

READING THE HOME TEST RESULTS

<p>ONE LINE APPEARS: the control line. The line can be light or dark.</p>	<p>TWO LINES APPEAR: the control line and the test line. One of the 2 lines may be lighter or darker than the other.</p>	<p>NO CONTROL LINE, NO TEST LINE.</p>	<p>IF YOUR HOME TEST DOES NOT MATCH ANY OF THE CASES SHOWN HERE, YOUR RESULT IS INVALID.</p>
			
<p>YOUR HOME TEST IS NEGATIVE</p>	<p>YOUR HOME TEST IS POSITIVE</p>	<p>YOUR HOME TEST IS INCONCLUSIVE OR INVALID</p>	
<p>YOU PROBABLY DO NOT HAVE COELIAC DISEASE</p> <p>The test shows that there are no IgA* anti-transglutaminase antibodies in the examined blood.</p> <p>The possibility of coeliac disease can be practically eliminated.</p> <p>If the gastro-intestinal pain continues see your doctor.</p>	<p>YOU PROBABLY HAVE COELIAC DISEASE</p> <ol style="list-style-type: none"> 1. SEE A DOCTOR as soon as possible and tell him/her that you have just taken the autotest GLUTEN® and your result is positive. 2. The result of your home test must be confirmed by a confirmation laboratory test. 3. You must not start a gluten-free diet before your doctor has received the results of the confirmation tests. He/she will confirm the diagnosis and prescribe the gluten-free diet. 	<p>SEE YOUR DOCTOR</p> <p>Two scenarios are possible:</p> <p>1st scenario: you may have a deficit of total IgA**.</p> <p>You cannot be detected by autotest GLUTEN®.</p> <p>2nd scenario: you may have made a handling error or the test may not have worked. Your test is therefore invalid.</p>	<p>Used lancets may be considered by your local authorities as medical waste.</p> <p>To reduce the risk of injury with used devices, follow the local recommendations for their disposal.</p> <p>Ask your pharmacist</p> <p>More information on autotest-sante.com</p>

*IgA: A Immunoglobulins. **The incidence of total IgA deficiency in the general population is approximately 0,5%. This deficiency is symptomless in most cases.

PRINCIPLE AND PERFORMANCES OF THE TEST

Celiac disease is an auto-immune disease of the intestine triggered by the consumption of gluten. It manifests as various symptoms : diarrhea, tiredness, abdominal pain, weight loss, anemia...If in doubt about the reason for your symptoms, have yourself screened and have the diagnosis confirmed by your doctor before starting a gluten-free diet or changing your eating habits.

autotest GLUTEN® is a home test for detecting celiac disease but the final diagnosis must be made, after a confirmation test, by a doctor.

In cases of celiac disease, a specific antibody is found in the patient's blood: the IgA anti-transglutaminase antibody. This is the marker that is detected and turns the T line positive.

False negatives (around 2% of celiac disease sufferers) found with both the usual laboratory tests and first-generation quick tests are due to a deficiency of total IgA. The second-generation autotest GLUTEN® was designed to correct this defect. This total IgA deficiency is indicated by the absence of the control line C. In this case, your doctor will prescribe a dose of IgG anti-transglutaminase in the laboratory.

The ESPGHAN (European Society for Paediatric Gastroenterology Hepatology and Nutrition) suggests that the dose of total IgA be given as first-line treatment. During a gluten-free diet, the level of celiac disease auto-antibodies will decrease and become undetectable at the latest 6 months after the change of diet. Consequently you will have a negative test result if you follow a gluten-free diet.

Thus autotest GLUTEN® is a good tool for monitoring the observance of a gluten-free diet under doctor's instructions among patients with celiac disease.

Test sensitivity and specificity study***: compared with the laboratory test, the sensitivity is of 95.7% and the specificity of 97.1%. The concordance of reading of the results of the quick tests was of 100% between two investigators. The test shows a positive predictive value (VPP) of 100% in the 44 positive patients with the quick test who agreed to undergo an intestinal biopsy. These patients all exhibited the lesions of the duodenal mucosa typical of celiac disease.

Reliability: the feasibility study of this home test by lay-persons showed that over 98% of participants who used this home test obtained an interpretable result.

***Study carried out on 150 patient samples compared with a laboratory test (I.RAIVIO et al. Alimentary pharmacology & Therapeutics).

PRECAUTIONS AND WARNINGS

- If the user instructions are not correctly followed, the test results may be false.
- If a gluten-free diet is already in progress, the test may give a false negative.
- A final diagnosis must be given only by a doctor, in conjunction with the clinical symptoms.
- The incidence of the total IgA deficiency is of approximately 2% among patients with the illness. Patients having a total IgA deficiency cannot be diagnosed by autotest GLUTEN®
- Do not begin a gluten-free diet without consulting a doctor.
- Do not use the test after the expiry date.
- Do not use the test if the foil pouch is damaged.
- Do not use damaged accessories.
- When the foil pouch is open, you must take the test within 60 minutes.
- The vial of diluent contains 0.09% sodium azide. Avoid contact with skin. Do not swallow!
- All the components of this test are designed for this test only. Do not reuse the test or accessories.
- Do not mix contents coming from different kits.
- Keep autotest GLUTEN® out of reach of children.

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