

117mm

Electric nasal aspirator Operating instruction

Method of use:

1.Before use,soak the mouthpiece in warm sated water for 10 minutes to disinfect,then instill the nasal aspirator according to the instructions,then turn on the switch,and start to see the nasal excrement from the nasal wall.

2.What do you do with dry boogers?Use the saline or the nasal spray that you buy frst,spray like your nostris,nasal spray should be fully exposed to the nasal cavity,and after four to five minutes.An invisible nasal discharge in the nasal cavity:

1.The suction nozzle does not touch the nose,you need to vacuum the vacuum principle to sniffle.

2.Let the baby stand or lie in half,head up(if the nose is dry,use saline or nasal spray to moisten the nasal cavity first).

3.Open the inhaler (adjustable fortwo gears),suction mouth about 45 degrees into the nostril with suction nozzle to completely block the nostris,forming a closed space(when the seal is in place,the machine voice becomes heavy).

4.Meanwhile,gently seal the other nostril with your finger,so that you don't have to seal it for a long time,so long as you seal off the nostril for 1-2 seconds.The nasal cavity forms anintegral sealing space,which operates several times until the noseis sucked out. Other common softening methods of nasal excretion:

Method 1:use sesame 1-2 drops to the baby inside nasal cavity.10~15minutes to soften dry nose,nasal absorption and then use the suck out nose excrement,cigarette mainly have the effect of a lubrication,this also has no side effects and is one of the traditional softening method,the doctor also suggested using time.

Method 2:apply the cotton swab with hot gauze or dip in hot water,apply it on both sides of the baby's nose bridge.and softenthe nasal excrement for 3~5 times in a row,and then use the inhaler.

Method 3:drop the liquid fish liveroil nto the baby's nose.Wait for 10-15 minutes,then suck out the nasal excrement with the nasal aspirator.

Please remember,do not use brute force or hard means to suck the baby's dry nose!

According to the customer feedback method 1 has the best effect and is the most traditional and no side effect.

Can't see the mucus,hear the nostril purring,how to solve?

Tip 1:use a baby cotton swab dipped in sesame oil gently to get him out,a lil bit toget a littlebit,and wait a few days for himself. Tips 2:a slight nasal congestion is used to cover the baby with a warm towel.The towel is to be used on the bridge of

139mm

the nose, and the towel is to be served. In a serious way, chop the ginger into the end, fry it in a pan, wrap it in gauze, and tie it to the baby's foot. Tip 3: it is to boil the onion with water, wet with a towel, twist dry (not too dry), put the baby's nose a few more times to do.

Move 4: use hot gauze or dip the cotton swab with hot water, apply on the two sides of baby's nose bridge, but how long, nose excrement will come out